

TauroPace™




CIED-placement

STEP BY STEP
INSTRUCTIONS

PREPARATION:


1

Surgical skin preparation must be carried out as close as possible to commencement of the procedure with an antiseptic solution recommended (consider fascial plane block for local anesthesia).

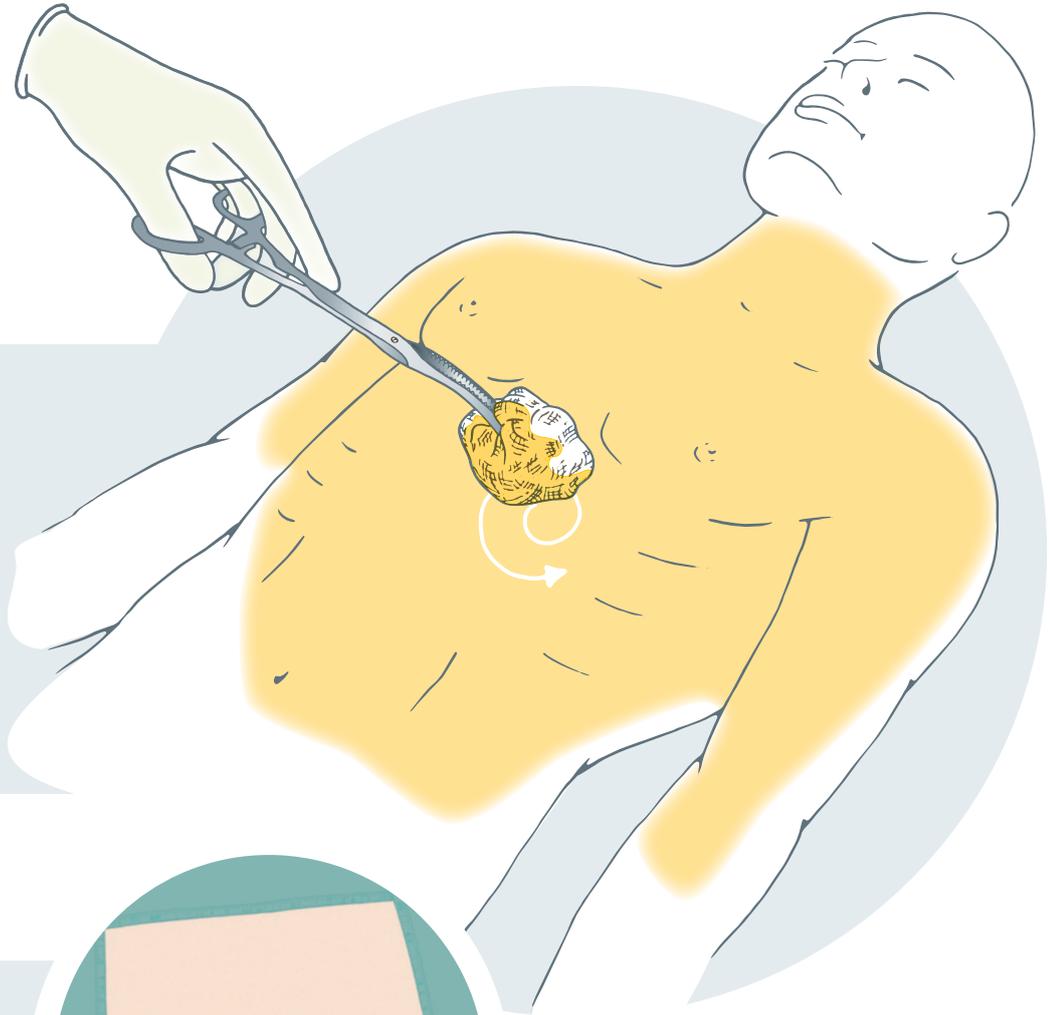


2

Drape the patient.

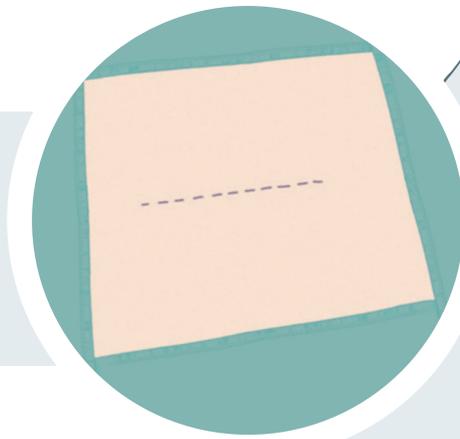

3

Transfer **TauroPace™** to a gallipot containing surgical swabs.

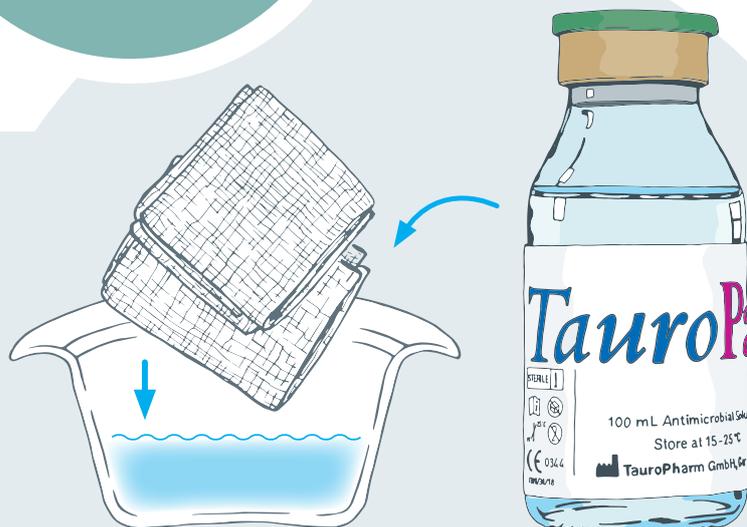


 TauroPace™

 antiseptic solution






5 ml

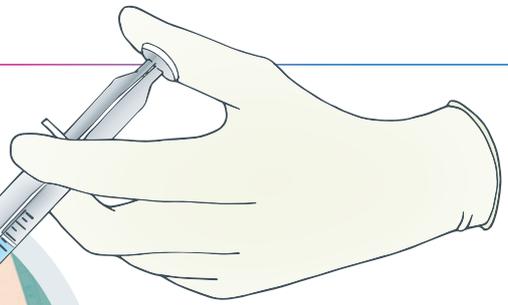


1

After skin incision and gaining access to/preparation of the vein wipe down sheath/dilator with **TauroPace™** before placement.

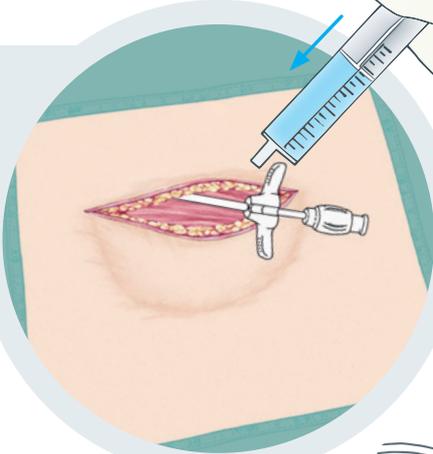

2

After placement rinse the surgical site with **TauroPace™**.

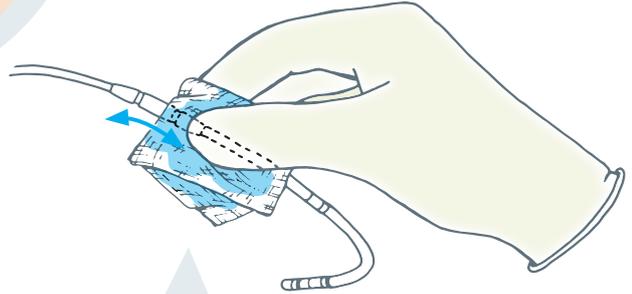


3

Wipe down the lead with a **TauroPace™** soaked swab prior to placement. Please ensure to mobilise the suture sleeve with a **TauroPace™** soaked swab (important).

4


Move the suture sleeve in final position with a **TauroPace™** soaked swab.

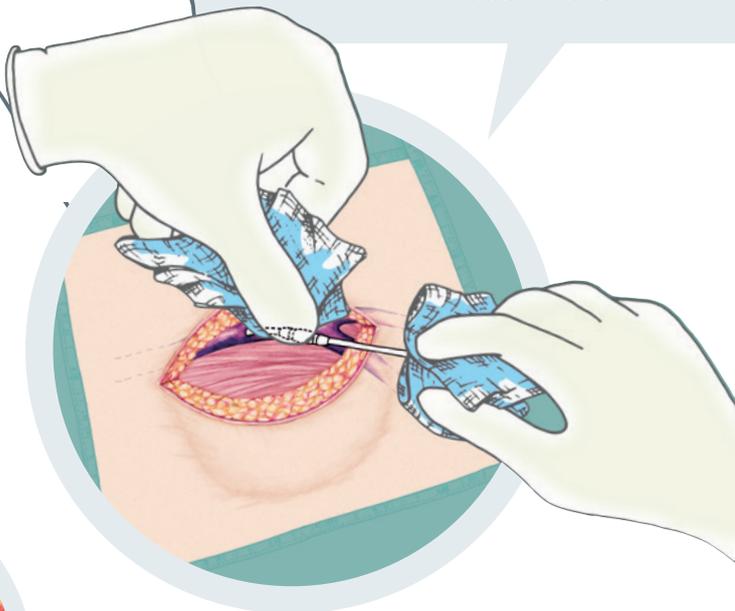


5


After placement of the lead(s), rinse the surgical site with **TauroPace™**.

5 ml

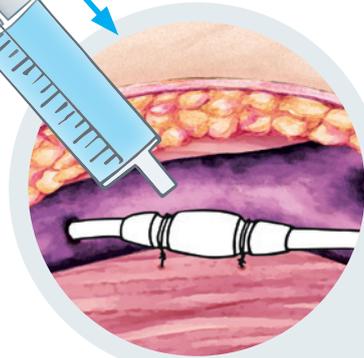


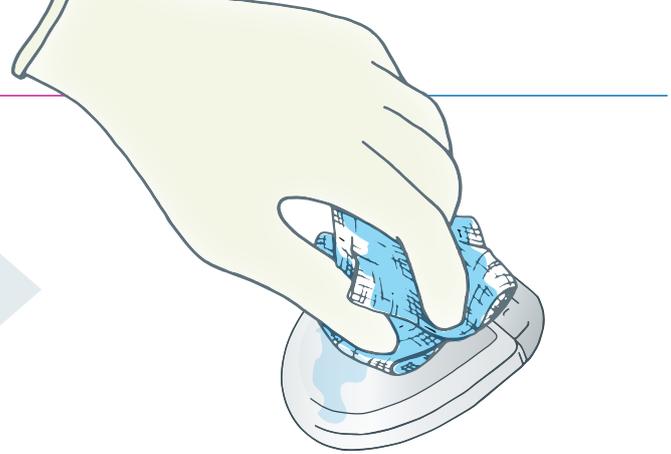


6 Wipe the new generator down with a **TauroPace™** soaked swab.

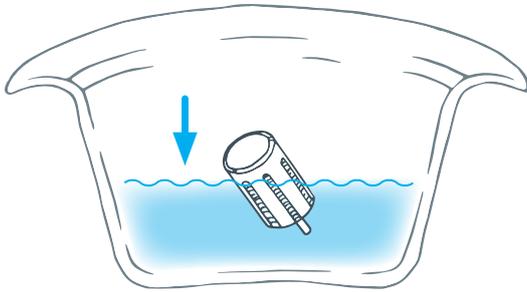

7 Immerse the torque wrench in **TauroPace™**.


8 Connect the lead(s) to the generator; use **TauroPace™** soaked swabs to hold on to the lead and the generator.

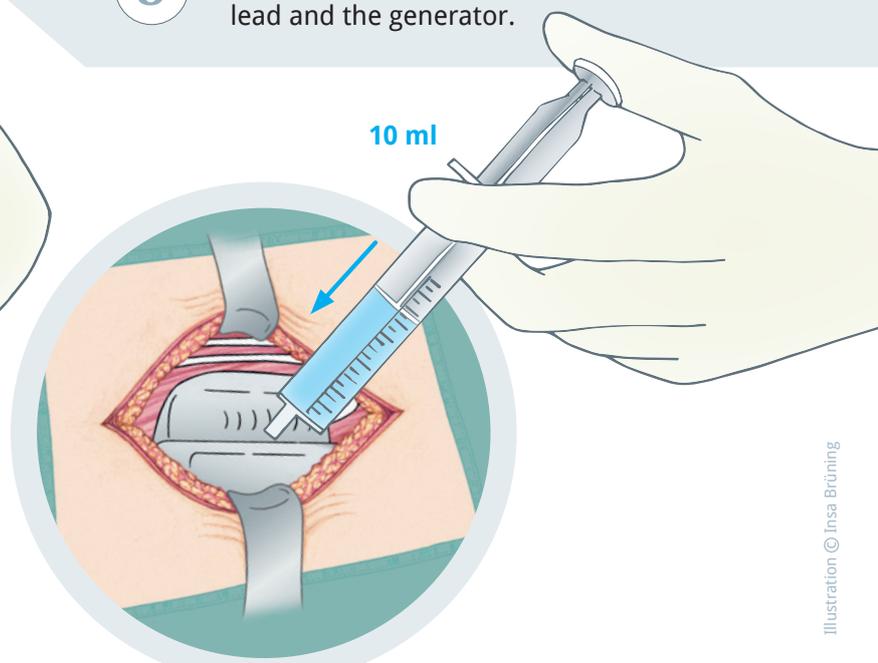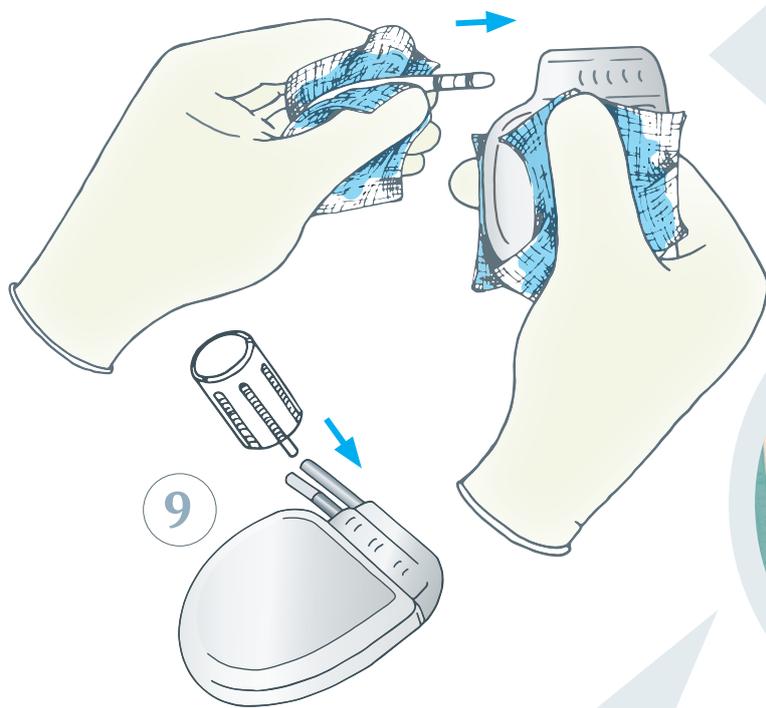

10 After final placement of all hardware, rinse the surgical site with **TauroPace™**.

11 Remove excess **TauroPace™**.

AFTER SKIN CLOSURE:

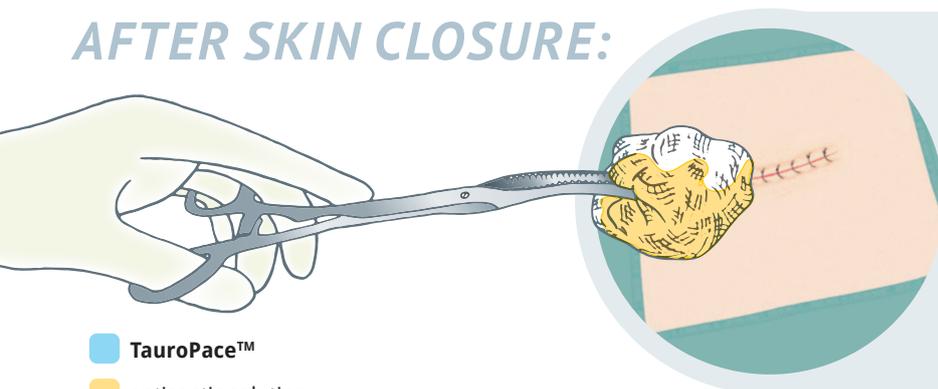

1 After skin closure wipe down with recommended skin antiseptic.

2 Apply a sterile wound dressing.

Use one bottle of **TauroPace™** per procedure.

TauroPace™
 antiseptic solution