


# TauroPace™

## S-ICD placement


### STEP BY STEP GUIDE

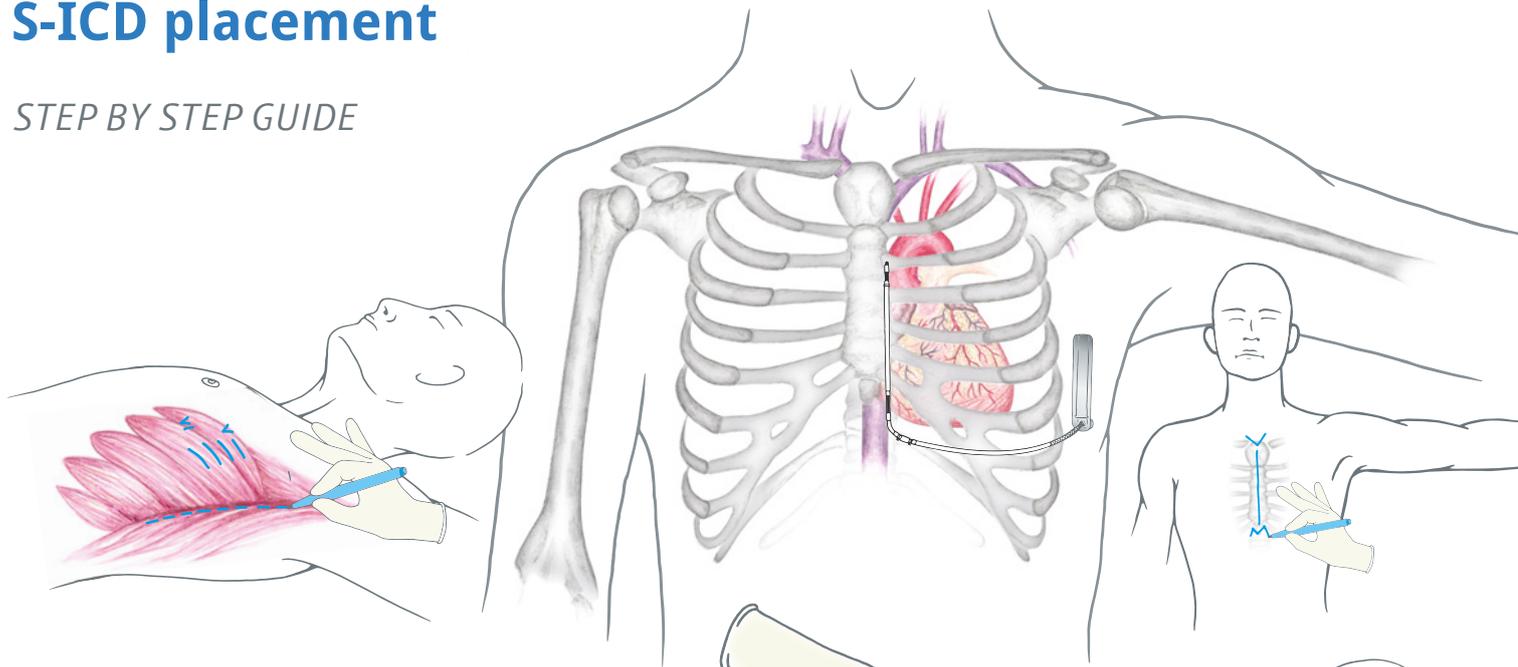

### PREPARATION:


1

Surgical skin preparation must be carried out as close as possible to commencement of the procedure with an antiseptic solution recommended (consider fascial plane block for local anesthesia).

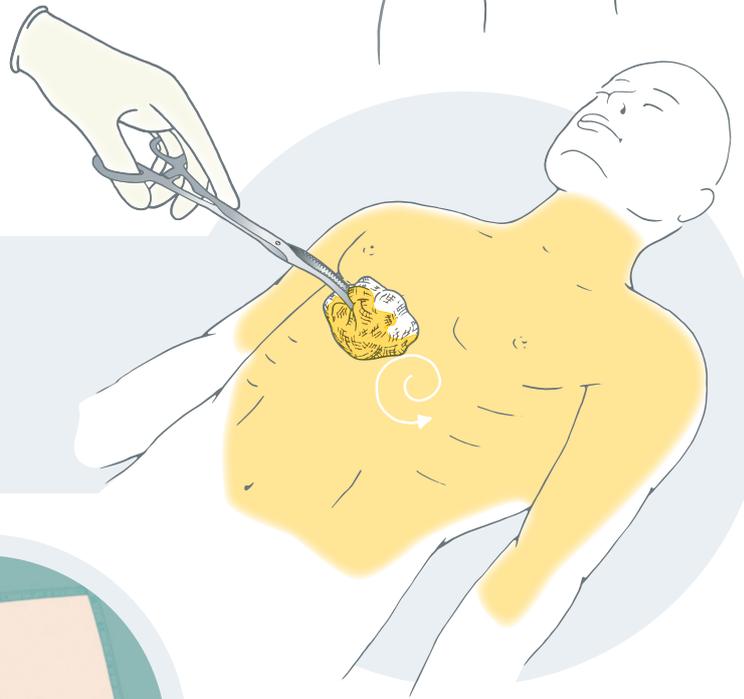


2

Drape the patient.

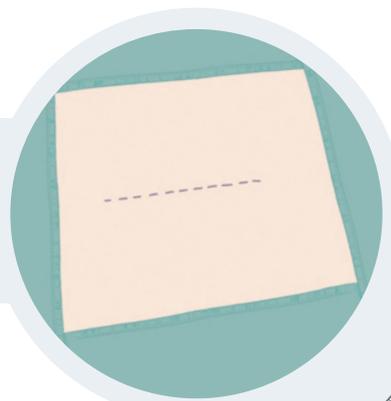

3

Transfer **TauroPace™** to a gallipot containing surgical swabs.

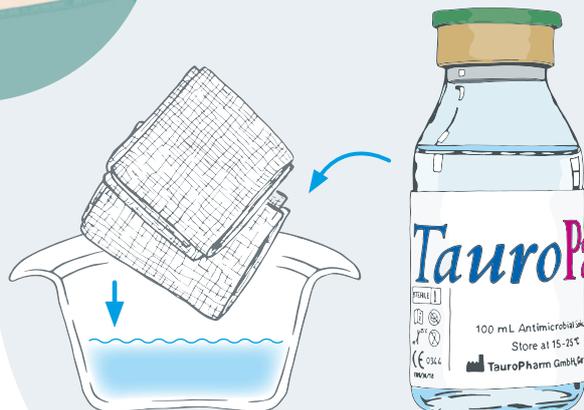


■ TauroPace™

■ antiseptic solution




1

Incise in the area of the caudal end of the xyphoid. Moisten the suture material with **TauroPace™** and apply.

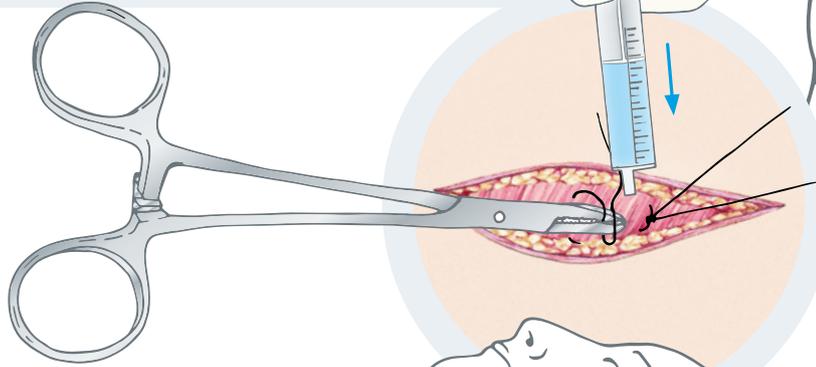


2





Prepare the generator pocket in an appropriate position. Moisten the suture material with **TauroPace™** and apply.

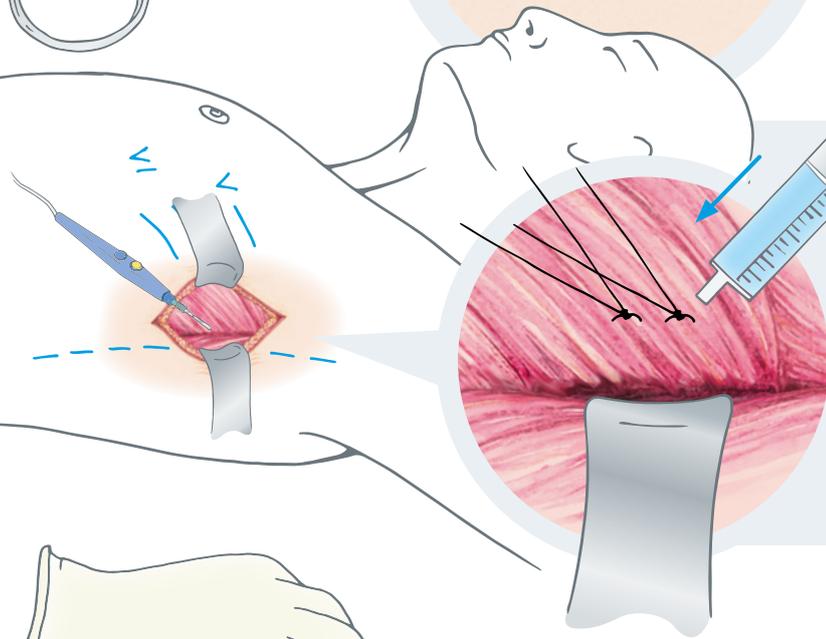

3

Tunnel with the decontaminated trocar from the incision in the xyphoid area towards the generator pocket, connect the lead to the caudal end of the trocar and pull the lead towards the xyphoid. While retracting the lead, wipe it down simultaneously with a **TauroPace™** soaked swab.

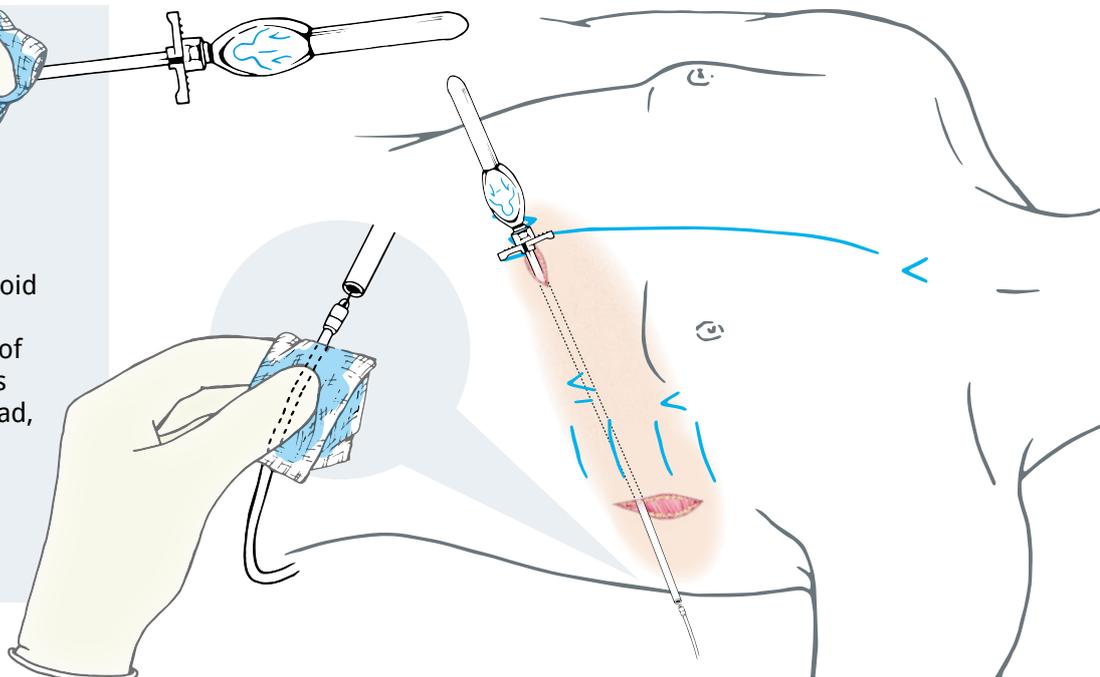

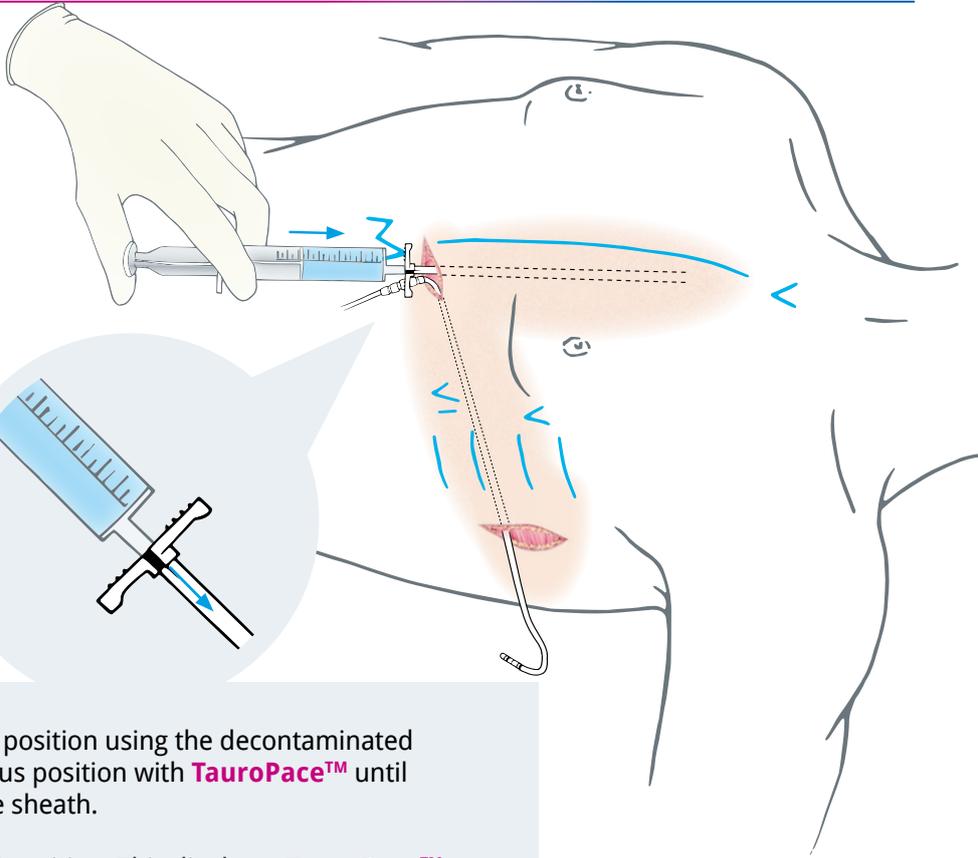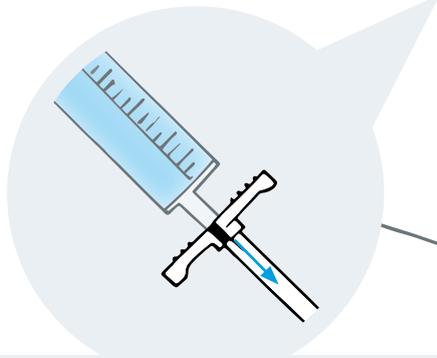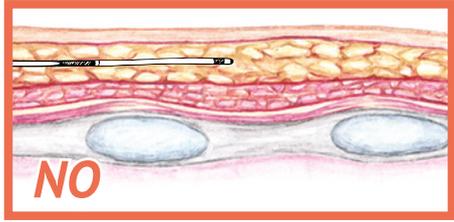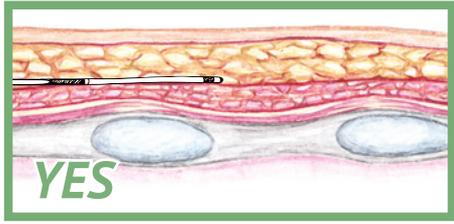


4

After inserting the sheath in sternal position using the decontaminated trocar, fill the sheath in subcutaneous position with **TauroPace™** until excess **TauroPace™** pours from the sheath.

Implant the shock coil in the sternal position. This displaces **TauroPace™** and prevents the implantation of subcutaneous air.


5

Fix the lead in this position to the fixation sleeve in the incision area (xyphoid) and moisten the components with **TauroPace™**. Peel away the sheath.

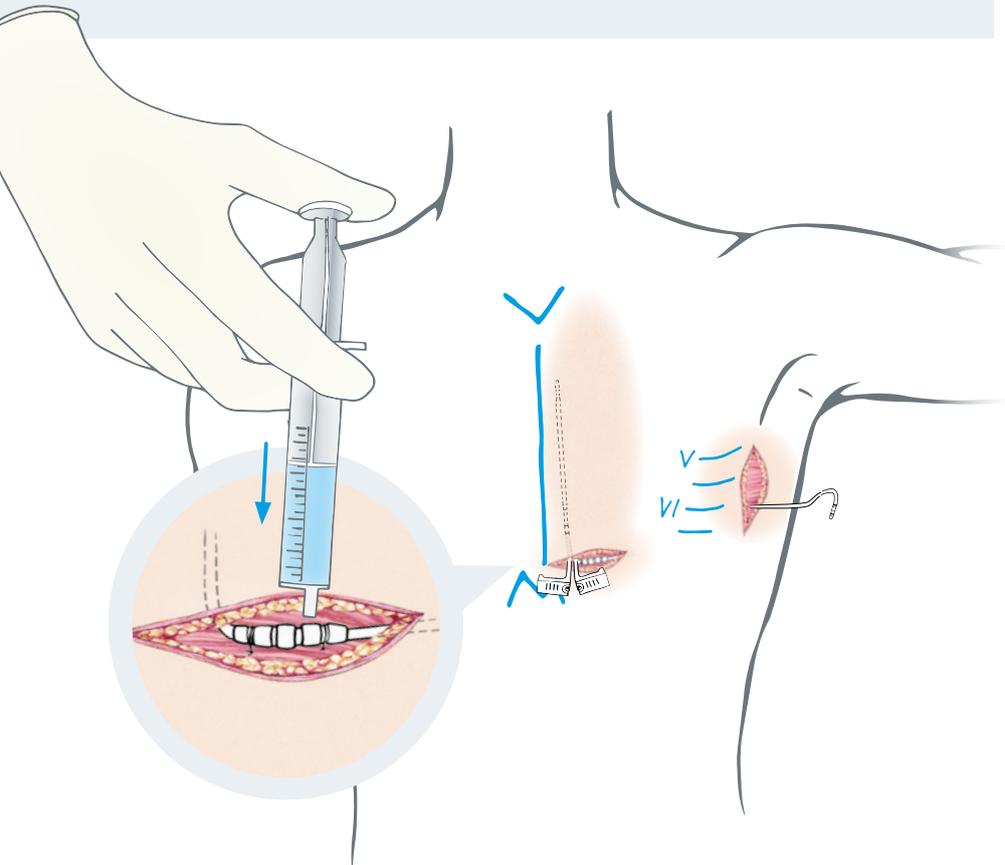


6

Wipe the new generator down with a **TauroPace™** soaked swab.

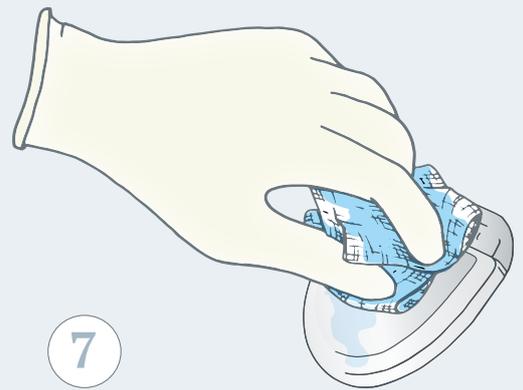

7

Immerse the torque wrench in **TauroPace™**.

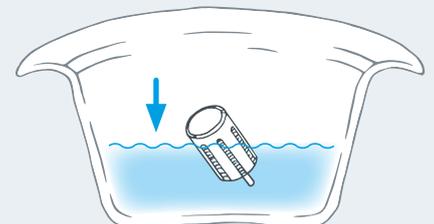



8

Connect the lead(s) to the generator; use **TauroPace™** soaked swabs to hold on to the lead and the generator.


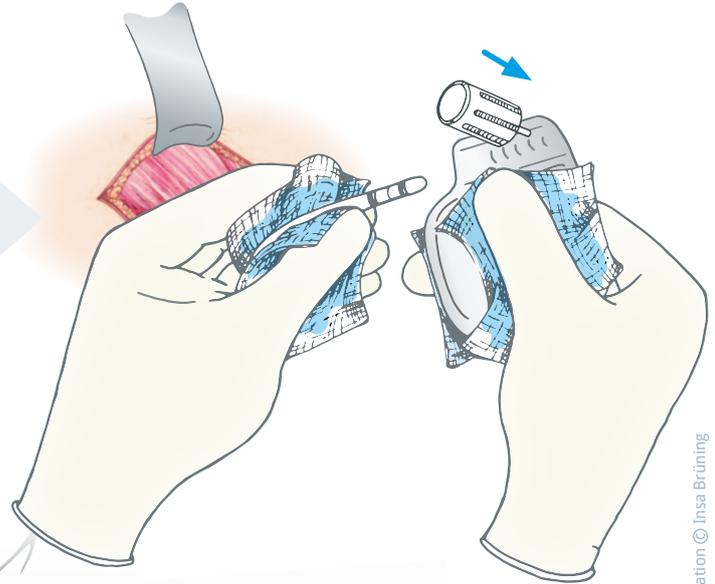


Illustration © Insa Brüning

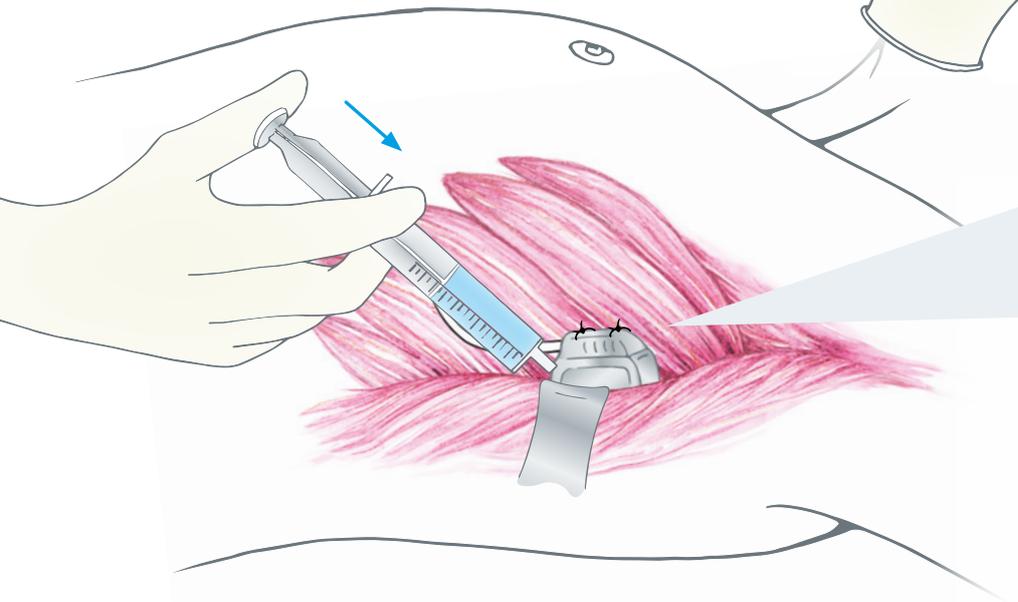

9

Take the remainders of **TauroPace™** and pool the pocket, the suture-fixed generator will position optimally.

10

Remove excess **TauroPace™**.

■ TauroPace™  
■ antiseptic solution

## AFTER SKIN CLOSURE:

1

After skin closure wipe down with recommended skin antiseptic.

2

Apply a sterile wound dressing.

Use one bottle of **TauroPace™** per procedure.

